(de)	Gebrauchsanweisung, VET-Otoskop	2
(en)	User's Manual, otoscope VET	7
(fr)	Mode d'emploi, otoscope VET	11
(it)	Istruzioni per l'uso, otoscopio VET	16
(es)	Instrucciones de empleo, otoscópio VET	21
(pt)	Manual de operação, otoscopio VET	26
(ru)	Руководство по применению, отоскоп VET	31

KaWe EUROLIGHT VET/ VET C30

CE

QM-1-131H

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User's Manual – Veterinary Otoscopes
KaWe EUROLIGHT VET C30 / VET C30 OP

Dear customer, thank you for choosing a KaWe product. Our products are known for their high quality and long life.

Please read these instructions thoroughly and carefully before attempting to use this product and heed the care instructions.

Familiarize yourself fully with this product before using it.

Use: These veterinary otoscopes are only to be used by authorized medical personnel. They are intended for the medical inspection of the natural body orifices such as the nose, ear, or oral cavities of animals.

Purpose: The veterinary otoscopes with attached ear funnel are used for the direct examination of the auditory canal and eardrum.

Warning: Proper operation is guaranteed only when standard 2.5V KaWe handles are used. These require two 1.5V "C" batteries (we recommend alkaline batteries) or KaWe-brand rechargeable batteries (2.5 V or 3.5V). Ensure that the polarity and type of bulb (2.5V or 3.5V) are correct! Attach the instrument head to the handle and lock it into place.

Operation: Turn on the lamp by pressing down and rotating the switch to the left. The light intensity (maximum switch position) can easily be adjusted with the rheostat. Turn off by rotating the switch back to the right to its starting position. To exchange the battery, unscrew the bottom cap. Handles powered by

rechargeable batteries (NiMH or Li-Ion) can be charged with the KaWe MedCharge 4000 charging station

Replacing the bulb: Twist the bulb out of the headpiece of the KaWe EUROLIGHT VET as directed by the arrow and remove the black plastic sleeve from the bulb. Screw in the new bulb and then slip the plastic sleeve back onto the bulb. The bulb must be clean and free of finger prints (grease). Ensure that the bulb has been fully inserted into the socket.

Unsuited use/contraindication of the products:

Any use other than that described here is not in accordance with the intended use of the unit. The manufacturer is not liable for any resulting damages. There are no known risks. The user alone bears the risk.

Further information, maintenance, storage:

When used and stored properly, these otoscopes will serve you many years.

Guarantee: When used under normal circumstances

and with attention to these instructions we guarantee this product for two years after the date of purchase (with the exception of bulbs and rechargeable batteries). Should you need further information or should your instrument require repair, please contact your dealer.

Warning instructions: When cleaning the reusable ear funnels, special attention is required. A temperature of 134°C in the autoclave is not to be exceeded.

When switched on for a long time the parts around the bulb can become very hot. As the otoscope is not intended for use during operational procedures, simple cleaning and or disinfection with an alcohol-based surface disinfecting agent is sufficient.

KaWe otoscopes should not be used in rooms where the temperature may exceed 35° C. KaWe otoscopes are subject to special safety measures regarding electromagnetic compatibility (EMC). The operation of the KaWe otoscope can be influenced by portable and mobile HF communication devices.

Limitation of reprocessing: Frequent reprocessing influences the instruments only minimally. The working life of the instruments usually depends on the wear and tear and damage by use. It is recommended, that the ear funnels be re-prepared as soon as possible after their use on a patient.

Instructions

Workstation: Remove dirt accumulation on surfaces with a paper/disposable cloth.

Storage and transport: No special requirements

Cleaning preparation: No special requirements

Manual cleaning: The exterior surface of the veterinary otoscope can be cleaned with a damp, soft, lint-free cloth. Never allow the otoscope to come into contact with fluids and ensure that liquids do not penetrate the housing. Do not use abrasive cleaning

agents! The unit cannot be sterilized. The magnifying glass is to only be cleaned with glass cleaner.

Automatic cleaning: No automatic cleaning possible/required.

Disinfection: Only the reusable ear funnels need to be disinfected with (for example) helipur. Follow the concentrations and reaction times given by the manufacturer. The veterinary otoscope can be disinfected by wiping it off with a cloth damped with alcohol.

Sterilization: Reusable funnels must be sterilized in an autoclave at a temperature of 134°C for a minimum of 3 minutes.

Additional information: If different kinds of instruments are being sterilized in the autoclave at the same time, the maximum load given by the manufacturer must not be exceeded.

Maintenance: Defective parts must be removed and replaced. If the device does not function properly, follow these steps: 1. Replace batteries/rechargeable batteries and check their functionality; 2. Replace the bulb and check its functionality; 3. Send the unit in for repair.

Check and performance test: Check all movable parts such as the frames of the magnifying glasses and plug-in connections to ensure that they all move smoothly. Turn on the unit. For all units: Visually check all parts for damage and wear.

Packaging: Individually packed in a cloth bag or box.

Storage: No special requirements.

Disposal: The product is to be disposed of at a separate waste disposal location in accordance with local ordinances.

User's Manual/Maintenance Instructions for Rechargeable NiMH/Li-Ion Batteries

Purpose: The rechargeable batteries are to be used as an energy source for the intended KaWe products.

Unsuited use/contraindication of the products:

Any use other than that described here is not in accordance with the intended use of this product. The manufacturer is not liable for any resulting damages. The user alone bears the risk.

Further information, maintenance, storage:

The charging batteries can be cleaned if necessary with a damp cloth and then wiped with a dry cloth. Please ensure that no short-circuit is produced.

The use of rechargeable batteries that are not explicitly approved for these units may cause explosions or leakages and the risk of fire, injuries and damage to surroundings. In the case that a rechargeable battery begins to leak and spilled substances come into contact with eyes, mouth, skin or clothing, immediately rinse the effected spot with water and consult a doctor.

Further maintenance instructions that may prolong the life of the rechargeable battery:

Please heed the following instructions: The non-observance of these instructions may result in the rechargeable battery losing its full capacity within a short period of time.

With NiMH technology, a lazy battery effect can occur that is analogous to the classic memory effect. It is

triggered by the energy that has not been completely spent prior to recharging the battery. This effect can be nearly eliminated by completely discharging the battery from time to time and then by recharging it two or three times in quick succession. Subsequently, the rechargeable battery will regain its full capacity. In the case that the rechargeable battery no longer provides enough energy to power the unit, immediately turn the unit off and charge the battery (in order to prevent it from losing its charge completely which would result in the destruction of the rechargeable battery).

Please keep in mind that the rechargeable batteries can spontaneously discharge. When the rechargeable batteries are not used (charged) for a longer period of time, a self-discharge may occur. The capacity will be lost and the rechargeable battery may become permanently defective. The manufacturer is not liable for any resulting damages. Therefore, when stored for more than one month, the rechargeable battery should be partially charged to approx. 50%.

Please always heed the given charging time in order to prevent overcharging and the resulting destruction of the rechargeable battery (the KaWe charging station KaWe MedCharge 4000 has an integrated automatic shut-off). NiMH rechargeable batteries lose their capacity at low temperatures (as low as 0°C).

Risks and notice of possible danger: **1.** Avoid short-circuiting the rechargeable batteries. **2.** Never allow the rechargeable batteries to come into contact with liquids. **3.** Keep the rechargeable battery away from heating installations and open flames. **4.** Rechargeable are not to be immersed in water or salt water **5.** Never try to disassemble or modify the rechargeable batteries or expose them to heat. **6.** Avoid dropping

the rechargeable batteries and protect them against being hit or crushed as this may damage the housing.

7. Please only use the rechargeable battery and accessory parts recommended by KaWe. **8.** When using a rechargeable battery for the first time, please make sure to charge it completely. **9.** When charging is completed and the charging unit and the compact adapter are no longer in use, please disconnect them from the unit and the power socket in order to avoid fires and other hazards. **10.** Do not cover the charging unit with any kind of material (tablecloths, carpets, bedclothes etc.) while operating the instrument. If you use the adapter for a longer period of time, it may become overheated, lose its original shape or catch fire. **11.** Please only charge the rechargeable battery in the appropriate rechargeable battery charging unit. The charging unit and the compact adapter have been developed especially for use with your unit. Please do not use them with other products or re-chargeable batteries. All other products may cause the risk of excessive heat and distortion and thus the risk of fire and electric shock. **12.** Prior to disposing of the batteries, ensure that the contacts are masked with adhesive strips or other insulating material in order to prevent their direct contact with other materials. Contact with other metallic material in trash cans may cause a fire or explosion.

Charging time:

REF	Capacity [mAh]	Charging time [h]
12.80110.722	3000	approx. 14*
12.80220.722	2200	approx. 10*
12.80120.742	700	approx. 12**

*Charging time after complete discharge of the rechargeable battery in the KaWe MedCharge 4000 charging station.

**Charging time after complete discharge of the rechargeable batteries in a wall electrical outlet.

at 30 % - 75% rel. humidity

Storage	+5°C to +45°C
Transport	-20°C to +50°C
Operation	+5°C to +40°C

Inserting batteries into the KaWe EUROLIGHT
2 x "C" batteries

Common accessories: Further information about these products can be obtained in our Internet shop at: www.kawemed.com

Manufacturer: **KaWe**

Contacting the manufacturer: Address or telephone number of the dealer or dial +49-7141-68188-0.

Accessories:

Reusable ear funnel: 4.0 / 5.0 / 7.0 mm

Symbol key:

	Manufacturer
	Date of manufacture
	Read the User's Manual
	Separate disposal of electric and electronic devices
	Dispose of expired batteries separately at an appropriate collection centre
	GOST-R certification for exports to Russia
	Keep dry
	Temperature limit

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KIRCHNER & WILHELM GmbH + Co. KG